

ALPRON - DAILY

Alpron is a liquid concentrate made for the continuous maintenance of the water quality in procedural water lines. Alpron is physiologically harmless and safe for procedural use. We would recommend use of Alpron in all water that runs through the chair to decrease the chance of biofilm formation in your waterlines.

Use throughout the day

The Alpron dosing is 10ml to 990ml, so roughly 1%. At the start of your day fill your procedural bottle with the appropriate dosage and use bottle contents throughout patients until empty and repeat.

Overnight

Alpron dosed procedural water is safe to leave in the bottle overnight as it is an active biocide that will prevent Biofilm from growing. However, it is recommended to at least a couple times a week, empty the contents of the bottle and leave it to dry in a suitable place.



BILPRON - WEEKLY / BI-WEEKLY

Bilpron is a ready to use liquid disinfectant that detaches and breaks down organisms inside the waterlines of your chair. Bilpron use is recommended weekly or bi-weekly as it is important to give the dental unit waterlines a shock treatment to decrease the chances of build-up of biofilm.

When to apply

The best time to use Bilpron is on a Friday afternoon as Bilpron requires a minimum of 12 hours in the unit waterlines to be effective and the longer you leave it, the better.

How to apply

Bilpron concentrate is not diluted when put into the chair, it only requires approximately 100ml in your bottle to flush through each handpiece and triplex line. Ensure that as soon as the blue indication of Bilpron comes out of the handpiece line you switch to the next line immediately to minimise wastage. Once the Bilpron has visibly been through each handpiece line you can switch the chair and bottle off. Remove the bottle, tip out remaining Bilpron from inside the bottle and leave to air dry over the period of the Bilpron flush. This prevents biofilm-growth on parts of the bottle and straw that aren't in direct contact with Bilpron by allowing them to be air purged. Returning to the chair after the shock treatment, fill the bottle with water and flush the water lines for 3-5 minutes.



**SCAN FOR
SUPPORT**

Equipment Sales, Service and Support

07 3139 1549 | sales@leadingdental.com.au